



YOU CAN DO IT

WITHOUT FINGERSTICKS*

Now it's even easier[†] to share your glucose data with your doctor.

CONNECT TO YOUR DOCTOR'S OFFICE WITH:

FreeStyle *LibreLink* 

AND UPLOAD GLUCOSE DATA TO:

LibreView



With the FreeStyle LibreLink app,^{†,§,||} the “Connect to a Practice” feature lets you easily:

- **CONNECT** to your doctor's LibreView[†] account with a one-time, easy setup
- **SHARE** your glucose data automatically[#] with your doctor in real time

With the secure LibreView[†] website, you and your doctor can:

- **VIEW AND SHARE** your glucose data in detailed reports
- **USE** the reports to guide informed treatment decisions



Get connected and start sharing today!

In the FreeStyle LibreLink app,[†] go to:
Menu > Share > LibreView


There are 2 ways to get started:

OPTION 1

Enter your doctor's LibreView Practice ID into the FreeStyle LibreLink app

Ask your doctor or their office staff to provide their LibreView Practice ID, which is a code specific to their practice.


My doctor's LibreView Practice ID is:

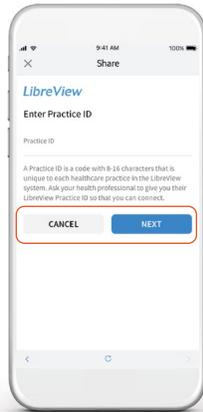

OPTION 2

Accept your doctor's invitation to connect

Your doctor's office will send you an email and in-app invitation to connect to their practice. Simply accept the invitation in the FreeStyle LibreLink app.[‡]

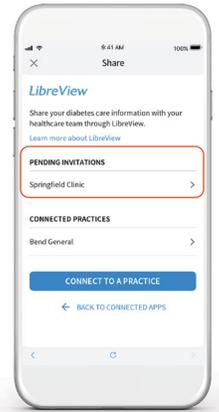

Questions? We're here for you.

1-855-632-8658

Available 7 days a week
8 AM to 8 PM Eastern Time, excluding holidays



* Fingersticks are required for treatment decisions when you see Check Blood Glucose symbol, when symptoms do not match system readings, when you suspect readings may be inaccurate, or when you experience symptoms that may be due to high or low blood glucose.

† Compared to uploading data to LibreView using the FreeStyle Libre 14 day reader.

‡ The FreeStyle LibreLink app is only compatible with certain mobile devices and operating systems. Please check the website for more information about device compatibility before using the app.

§ The FreeStyle LibreLink app and the FreeStyle Libre 14 day reader have similar but not identical features. Fingersticks are required for treatment decisions when you see Check Blood Glucose symbol, when symptoms do not match system readings, when you suspect readings may be inaccurate, or when you experience symptoms that may be due to high or low blood glucose.

|| Medicare coverage is available for FreeStyle Libre 14 day systems for cell phone use if FreeStyle LibreLink is used in conjunction with the FreeStyle Libre and FreeStyle Libre 14 day readers. Patients must meet Medicare eligibility coverage criteria. Local Coverage Article: Glucose Monitor Policy Article (A52464), May 2017.

¶ Use of the FreeStyle LibreLink app requires registration with LibreView, a service provided by Abbott and Newyu, Inc.

The user must be connected to data services for glucose data to automatically upload to LibreView.

Indications and Important Safety Information

FreeStyle Libre 14 day Flash Glucose Monitoring system is a continuous glucose monitoring (CGM) device indicated for replacing blood glucose testing and detecting trends and tracking patterns aiding in the detection of episodes of hyperglycemia and hypoglycemia, facilitating both acute and long-term therapy adjustments in persons (age 18 and older) with diabetes. The system is intended for single patient use and requires a prescription.

CONTRAINDICATIONS: Remove the sensor before MRI, CT scan, X-ray, or diathermy treatment.

WARNINGS/LIMITATIONS: Do not ignore symptoms that may be due to low or high blood glucose, hypoglycemic unawareness, or dehydration. Check sensor glucose readings with a blood glucose meter when Check Blood Glucose symbol appears, when symptoms do not match system readings, or when readings are suspected to be inaccurate. The systems do not have alarms unless the sensor is scanned, and the systems contain small parts that may be dangerous if swallowed. The systems are not approved for pregnant women, persons on dialysis, or critically-ill population. Sensor placement is not approved for sites other than the back of the arm and standard precautions for transmission of blood borne pathogens should be taken. The built-in blood glucose meter is not for use on dehydrated, hypotensive, in shock, hyperglycemic-hyperosmolar state, with or without ketosis, neonates, critically-ill patients, or for diagnosis or screening of diabetes. When using FreeStyle LibreLink app, access to a blood glucose monitoring system is required as the app does not provide one. Review all product information before use or contact Abbott Toll Free (855-632-8658) or visit www.FreeStyleLibre.us for detailed indications for use and safety information. For full indications for use and safety information, see more at www.freestylelibre.us/safety-information.html.

FreeStyle, Libre, and related brand marks are trademarks of Abbott Diabetes Care Inc. in various jurisdictions. Other trademarks are the property of their respective owners. Product images are for illustrative purposes only.

©2020 Abbott. All rights reserved. ADC-20465 v1.0 4/20